



We put space to work.

Item # _____

Job _____

CaseVue™ Case Carts & Accessories

One look and the advantage is clear.

Built for easy transport, the CaseVue™ case cart is stronger, lighter, and the most ergonomic choice available. Carts can be used wherever contents must be protected from an “unclean” environment, including hospital based or offsite surgery centers. Advantages provided by the patent pending CaseVue case carts include:

Proficient:

Maintenance-free casters reduce the total cost of ownership. Optional clear doors and status flags provide a quick view of cart's status at any time. Cool touch handles eliminate the need for protective cloth to remove cart from cart wash. Biohazard wheel indicates used carts and paper holder conveniently holds pick lists and routing slips.

Clean Design:

Cart is designed to be easily cleaned and not retain water. Stainless steel corner handles are encased in a cool touch polymer featuring Microban® antimicrobial product protection.

Less Effort:

Premier, quiet, maintenance-free casters reduce rolling resistance; wide grip handles provide better leverage for steering and swivel lock casters keep the cart on track. Optional upgrades, including 5th wheel steering and swing up handles, provide maximum control in transit.

Quiet:

Contoured sides, reinforced top and floor, softer wheels and other noise dampeners, provide a noticeably quieter ride.

Durable:

Type 304 stainless steel construction, including stainless steel casters are standard, not an optional upgrade.

Efficient:

Metro CaseVue delivers the most storage per square foot to ensure we are making the most of your space. Roller shelves extend further to provide better access.

Configurable:

CaseVue provides many standard items to customize the cart to your needs, including clear doors, various sizes, solid or wire roller shelves, 5th wheel steering, colored status flags and swing-up handles.



CV36LC-5BFH
Shown with 1 CV2436S



CV24HS-NBFH



CaseVue™ Case Carts & Accessories



InterMetro Industries Corporation
North Washington Street, Wilkes-Barre, PA 18705
Product Information. U.S. and Canada: 1.800.992.1776
Outside U.S. and Canada: www.metro.com/contactus

LO2-250
Printed in U.S.A. 6/22

Information and specifications are subject to change without notice. Please confirm at time of order.

Copyright © 2022 InterMetro Industries Corp.

33.11



Specifications

Cart Body:

- Type 304 Stainless Steel, No. 3 finish type
- Contoured side panels rigidize body and dampen vibration noise
- Four type 304 stainless steel, wide grip handles encased in injection molded flexible PVC with Microban antimicrobial product protection
- Sloped flanges to promote water drainage
- Raised front on floor to retain spills in transit
- Reinforced top and floor to stiffen surface and reduce vibration noise
- Stainless steel door stay with impact-resistant polymer catch
- Polymer and stainless steel roll pin paper holder

Doors:

- Type 304 stainless steel single panel
- Optional: polycarbonate clear window sealed with rubber gasket
- Sloped flanges to promote drainage
- Replaceable paddle latch with security option
- Rubber bumpers to reduce rattle noise

Base Assembly:

- Replaceable non marking, type 304 stainless corner bumpers encased in flexible PVC, with front and rear continuous bumper
- Replaceable type 304 stainless steel caster channels

Shelves

- Roller pullout stainless steel wire or solid
- Built in retainer that holds shelf closed during cart wash
- Each shelf includes two shelf brackets with nylon rollers
- Shelf bracket include a shelf stop to prevent over extension
- Adjustable on 2.25" (57mm) increments
- Rubber bumpers to minimize rattle noise

Swing Up Extended Handle

- Type 304 stainless steel tubular construction
- Swings up to desired height of user
- Closes within footprint of the cart
- Rubber bumpers reduce rattle noise

Available options

(Minimums may apply):

- Extended tops
- Tote boxes
- Custom sizes
- Tow assemblies

Casters:

Cart washable, Quiet, Easy Rolling, Maintenance-Free

- 6" (152mm) diameter, type 304 stainless steel
- Two total-lock casters on front of cart
- Two swivel-lock on rear for carts without 5th wheel steering
- Two swivel on rear for carts equipped with 5th wheel steering
- All stainless steel hardware and top plate
- Swivel bearing: Double race of stainless steel ball bearings, grease sealed, dust cap.
- Wheel bearing: Sealed precision ball bearing.
- Gray thermoplastic rubber (TPR) on gray polypropylene hub with thread guard; 1¼" (32mm) rounded tread face provides a softer, quieter ride. TPR resists flat spotting and repels floor debris.

5th Wheel Steering

- Available on select carts
- 5" (127mm) diameter, type 304 stainless steel
- Type 304 stainless steel mechanism with vinyl dipped foot activated lever - Two rear caster

Biohazard Indicator Wheel

- Type 304 stainless steel housing and wheel
- Wheel rotates and locks into position indicating Biohazard in red, or Clean in green

Color Status Flags

- 4 Injection molded ABS flags in red, green, orange and blue
- Stainless steel housing
- Flags positively held in up or down position
- Default color code (user definable)
 - Green: Cart fully picked and ready
 - Orange: Cart picked, OR supply needed
 - Red: Biohazard, soiled contents inside
 - Blue: Clean cart return (i.e. case cancelled)

Metro Closed Case Carts are cart washable. The cart wash chamber must be non pressurized. Water temperatures in the range of 140°-180°F (60°-82°C) are allowed with an exposure period not to exceed 30 minutes. Hot air drying is allowed with the temperatures not to exceed 200°F (93°C) and an exposure period not to exceed 15 minutes. Contact your Metro representative if cart wash parameters fall outside of above specifications.




Case Cart with Factory Installed Accessories

CaseVue standard features include, all stainless steel construction, cool touch antimicrobial wide grip handles, stainless steel casters, security door latch, picklist/paper holder, door catch and optional 5th wheel steering (available on 36 & 48 models).

Model Number Identifier:

CV36LS-5BFH

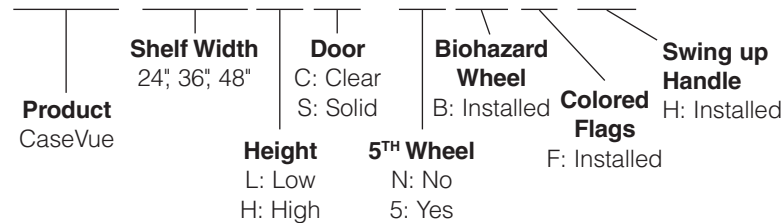

Models ending with **B, F,** or **H** include corresponding accessories factory assembled and ready to go out of the box.

B: Biohazard indicator wheel

F: Color flags

H: Swing up handle

NOTE: Accessories ordered with but not included in the CaseVue model number will be shipped separate from the cart and require onsite assembly to the cart.

Clear Door Carts

Carts with Factory Installed Accessories and 5th Wheel

Model. No.	Description	D x W x H	D x W x H
CV36LC-5BFH	Case Cart, Low, Clear Doors, 5th Wheel, Accessories	28 x 45½ x 40%	711 x 1156 x 1032
CV48LC-5BFH	Case Cart, Low, Clear Doors, 5th Wheel, Accessories	28 x 57½ x 40%	711 x 1461 x 1032
CV36HC-5BFH	Case Cart, High, Clear Doors, 5th Wheel, Accessories	28 x 45½ x 58¼	711 x 1156 x 1480

Carts with Factory Installed Accessories

Model. No.	Description	D x W x H	D x W x H
CV24LC-NBF	Case Cart, Low, Clear Door, Accessories	28 x 33½ x 40%	711 x 851 x 1032
CV36LC-NBFH	Case Cart, Low, Clear Doors, Accessories	28 x 45½ x 40%	711 x 1156 x 1032
CV48LC-NBFH	Case Cart, Low, Clear Doors, Accessories	28 x 57½ x 40%	711 x 1461 x 1032
CV24HC-NBFH	Case Cart, High, Clear Door, Accessories	28 x 33½ x 58¼	711 x 851 x 1480
CV36HC-NBFH	Case Cart, High, Clear Doors, Accessories	28 x 45½ x 58¼	711 x 1156 x 1480



CV24LC-NBF
Shown with (1) CV2424W wire shelf

Solid Door Carts

Carts with Factory Installed Accessories and 5th Wheel

Model. No.	Description	D x W x H	D x W x H
CV36LS-5BFH	Case Cart, Low, Solid Doors, 5th Wheel, Accessories	28 x 45½ x 40%	711 x 1156 x 1032
CV48LS-5BFH	Case Cart, Low, Solid Doors, 5th Wheel, Accessories	28 x 57½ x 40%	711 x 1461 x 1032
CV36HS-5BFH	Case Cart, High, Solid Doors, 5th Wheel, Accessories	28 x 45½ x 58¼	711 x 1156 x 1480

Carts with Factory Installed Accessories

Model. No.	Description	D x W x H	D x W x H
CV24LS-NBF	Case Cart, Low, Solid Door, Accessories	28 x 33½ x 40%	711 x 851 x 1032
CV36LS-NBFH	Case Cart, Low, Solid Doors, Accessories	28 x 45½ x 40%	711 x 1156 x 1032
CV48LS-NBFH	Case Cart, Low, Solid Doors, Accessories	28 x 57½ x 40%	711 x 1461 x 1032
CV24HS-NBFH	Case Cart, High, Solid Door, Accessories	28 x 33½ x 58¼	711 x 851 x 1480
CV36HS-NBFH	Case Cart, High, Solid Doors, Accessories	28 x 45½ x 58¼	711 x 1156 x 1480




CV36LS-5BFH

Customer-Installed Accessories

Wire Roller Shelves

Model. No.	Description	D x W x H	D x W x H
CV2424W	Shelf, Wire, Roller, for Case Cart	24 x 24	610 x 610
CV2436W	Shelf, Wire, Roller, for Case Cart	24 x 36	611 x 914
CV2448W	Shelf, Wire, Roller, for Case Cart	24 x 48	612 x 1219

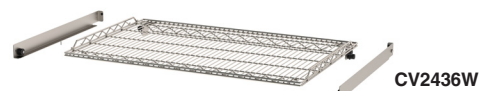

CV2436W

Solid Roller Shelves

Model. No.	Description	D x W x H	D x W x H
CV2424S	Shelf, Solid, Roller, for Case Cart	24 x 24	610 x 610
CV2436S	Shelf, Solid, Roller, for Case Cart	24 x 36	611 x 914
CV2448S	Shelf, Solid, Roller, for Case Cart	24 x 48	612 x 1219



CV2436S


Shelf weight capacity is 125lbs. (57kg) evenly distributed. Each shelf includes two attachment roller brackets with hold closed feature.



CaseVue Case Cart - Basic Models

CaseVue standard features include, all stainless-steel construction, cool touch antimicrobial wide-grip handles, stainless steel casters, security door latch, picklist/paper holder, door catch and optional 5th wheel steering (available on 36 & 48 models).

Model Number Identifier:

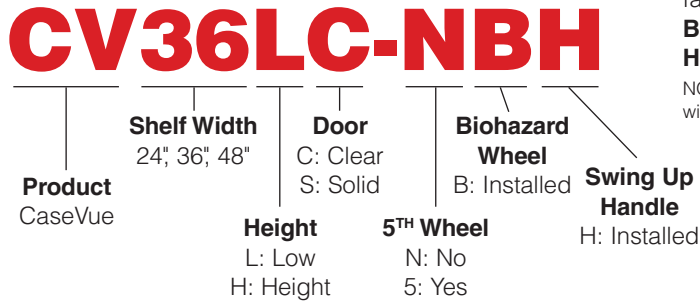

Models ending with **B** or **H** include corresponding accessories factory assembled and ready to go out of the box.

B - Biohazard Indicator Wheel

H - Swing Up Handle.

NOTE: Accessories ordered with but not included in the CaseVue model number will be shipped separate from the cart and require onsite assembly to the cart.

Clear Door Carts

Carts with Bio-wheel and 5th Wheel (Swing up Handle included on tall carts)

Model. No.	Description	D x W x H	D x W x H
CV36LC-5B	Case Cart, Low, Clear Doors, 5th Wheel, Bio-wheel	28 x 45½ x 40%	711 x 1156 x 1032
CV48LC-5B	Case Cart, Low, Clear Doors, 5th Wheel, Bio-wheel	28 x 57½ x 40%	711 x 1461 x 1032
CV36HC-5BH	Case Cart, High, Clear Doors, 5th Wheel, BW, Handle	28 x 45½ x 58¼	711 x 1156 x 1480

Carts with Bio-wheel (Swing up Handle included on tall carts)

Model. No.	Description	D x W x H	D x W x H
CV24LC-NB	Case Cart, Low, Clear Doors, Bio-wheel	28 x 33½ x 40%	711 x 851 x 1032
CV36LC-NB	Case Cart, Low, Clear Doors, Bio-wheel	28 x 45½ x 40%	711 x 1156 x 1032
CV48LC-NB	Case Cart, Low, Clear Doors, Bio-wheel	28 x 57½ x 40%	711 x 1461 x 1032
CV24HC-NBH	Case Cart, High, Clear Doors, BW, Handle	28 x 33½ x 58¼	711 x 851 x 1480
CV36HC-NBH	Case Cart, High, Clear Doors, BW, Handle	28 x 45½ x 58¼	711 x 1156 x 1480



CV48LC-NB

Shown with CV2448S shelf

Solid Door Carts

Carts with Bio-wheel and 5th Wheel (Swing up Handle included on tall carts)

Model. No.	Description	D x W x H	D x W x H
CV36LS-5B	Case Cart, Low, Solid Doors, 5th Wheel, Bio-wheel	28 x 45½ x 40%	711 x 1156 x 1032
CV48LS-5B	Case Cart, Low, Solid Doors, 5th Wheel, Bio-wheel	28 x 57½ x 40%	711 x 1461 x 1032
CV36HS-5BH	Case Cart, High, Solid Doors, 5th Wheel, BW, Handle	28 x 45½ x 58¼	711 x 1156 x 1480

Carts with Bio-wheel (Swing up Handle included on tall carts)

Model. No.	Description	D x W x H	D x W x H
CV24LS-NB	Case Cart, Low, Solid Doors, Bio-wheel	28 x 33½ x 40%	711 x 851 x 1032
CV36LS-NB	Case Cart, Low, Solid Doors, Bio-wheel	28 x 45½ x 40%	711 x 1156 x 1032
CV48LS-NB	Case Cart, Low, Solid Doors, Bio-wheel	28 x 57½ x 40%	711 x 1461 x 1032
CV24HS-NBH	Case Cart, High, Solid Doors, BW, Handle	28 x 33½ x 58¼	711 x 851 x 1480
CV36HS-NBH	Case Cart, High, Solid Doors, BW, Handle	28 x 45½ x 58¼	711 x 1156 x 1480



CV36HS-NBH

CaseVue™ Case Carts & Accessories

We put space to work.

CaseVue Field Installed Accessories

CaseVue Cart Accessories ordered with but not included in the CaseVue model number will be shipped separate from the cart and require onsite assembly to the cart.

Shelves

Stainless steel pullout shelves include two roller brackets that feature a built-in retainer that holds shelf closed during cart wash and a shelf stop that prevents over extension.

Wire Roller Shelves


Model. No.	Description	D x W x H	D x W x H
CV2424W	Shelf, Wire, Roller, for Case Cart	24 x 24	610 x 610
CV2436W	Shelf, Wire, Roller, for Case Cart	24 x 36	611 x 914
CV2448W	Shelf, Wire, Roller, for Case Cart	24 x 48	612 x 1219



CV2436W

Solid Roller Shelves

Model. No.	Description	D x W x H	D x W x H
CV2424S	Shelf, Solid, Roller, for Case Cart	24 x 24	610 x 610
CV2436S	Shelf, Solid, Roller, for Case Cart	24 x 36	611 x 914
CV2448S	Shelf, Solid, Roller, for Case Cart	24 x 48	612 x 1219

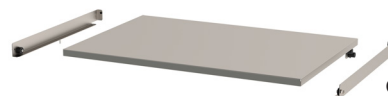

CV2436S

Shelf weight capacity is 125lbs. (57kg) evenly distributed.
Each shelf includes two attachment roller brackets with hold closed feature.

Cart Accessories

Model. No.	Description
CVFLAGS	Color Flags: Red, Orange, Blue, Green, w/ Hardware
CVHANDLE	Swing up handle, with hardware
CVBIO-RF*	Biohazard Indicator Wheel, Retrofit, w/ High-temp double-sided adhesive
CVSEAL24L	Door Seal for Low 24 Case Cart
CVSEAL24H	Door Seal for High 24 Case Cart
CVSEAL36L	Door Seal for Low 36 Case Cart
CVSEAL36H	Door Seal for High 36 Case Cart
CVSEAL48L	Door Seal for Low 48 Case Cart

*CVBIO-RF is designed to be field installed with screws to CaseVue solid door, or adhered with high temperature double sided tape to alternate location on any cart.



CVFLAGS



CVHANDLE



CVBIO-RF
Clean View



CVBIO-RF
Biohazard View