FLEXLINE ELECTRONIC PIN LOCK

DESCRIPTION

Flexline Electronic PIN Lock: This instruction sheet covers installation procedure, user guide for CompX Regulator, unlock/lock using key override and Electronic PIN Lock.

CompX[®] Regulator[®] USER GUIDE

The Regulator is used for locking cart drawers. It can mount to the cart top. Powered by 6-AA batteries (included), Regulator operates using a 4 to 8 digit code. Operation is simple: type a valid code and press **ENTER**. Then just turn the lever to the locked or unlocked position.

Regulator Lock offers:

- Programmable user and supervisor codes,
- Programmable modes of operation,
- Programmable reset features,
- Regulator is intended for interior use only,
- **Operating Temperature:** 50°F – 120°F (10°C – 48°C).

Each lock contains 2 green stickers that are needed for resetting Regulator. In the event Regulator needs to be reset, one green sticker is placed on the back of the device. The remaining green sticker should be saved in a safe and secure location for future reference. (See "**Reset Instructions**" on page #4.)

Care and Cleaning


To clean the Regulator surface, apply non-acid based cleaner to a clean cotton cloth; do not spray cleaner directly on the Regulator. Wipe down keypad and its respective housing surfaces of excess cleaner.

Powering Up and Battery Installation

CompX[®] Regulator[®] is shipped with six AA batteries. The **yellow light** on keypad of lock will indicate need for replacement. The Batteries can be replaced by removing the top two drawers and the retainer bracket. Refer the steps below for battery replacement.

NOTE: Mixing batteries of different chemistries could damage the Regulator resulting in lock failure.

Battery Replacement: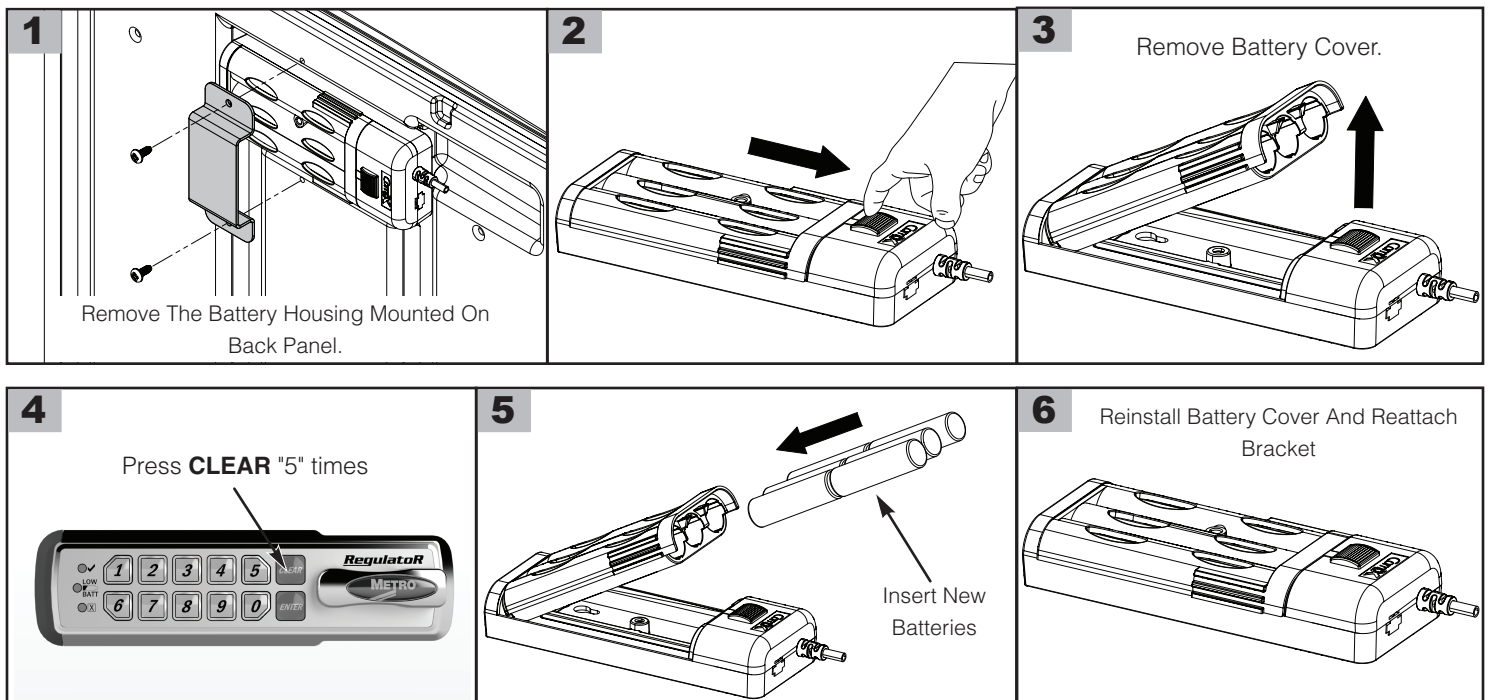

Programming – Introduction


CompX[®] Regulator[®] is easy to program and operate.

Type the user code (factory default 1234) and press **ENTER**, then rotate the knob clockwise to unlock the cart.

The supervisor can change operating codes, modes and restore lock settings.

While Regulator is easy to program, we highly recommend reading the complete programming instructions prior to making any changes. Remember to keep track of any programming changes.

Button locations, indicator lights and knob referenced in the programming instructions are shown below:

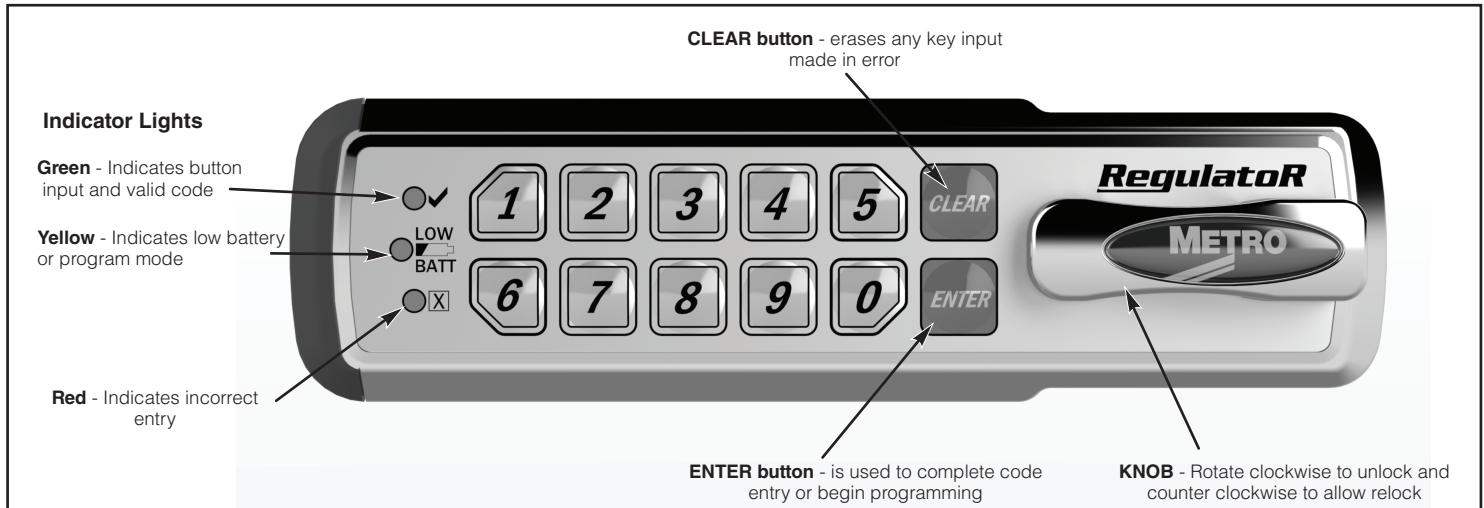Follow the directions below to operate (unlock and lock) the standard Self Lock version of RegulatorR:

- Type a valid code, press ENTER,
- Rotate knob clockwise,
- Release knob, RegulatorR knob will return to the locking position.

IMPORTANT: The Flexline cart is not secure until the red locking handle is pushed down to relock the cart !

RegulatorR Programming Mode

To enter programming mode you must know the supervisor code. The default supervisor code is 12345. Once in programming mode, multiple changes can be made prior to exiting programming.

NOTE: There is a 10 second programming time limit between each step. If RegulatorR times out or an error is made, programming will end and you will need to enter programming mode again.

1. Lock must be in the locked position prior to entering programming mode.
2. Type supervisor code and press **ENTER**.
3. Rotate the knob to the **unlocked** position and press and hold **ENTER** for "3 seconds" (yellow indicator light will illuminate).
4. You can now let go of the knob.
5. With the **yellow** indicator light illuminated, programming can begin.

Changing the User Code

To change the user code, refer to "**RegulatorR Programming Mode**" to enter programming mode. The factory default user code is 1234 (User codes must be 4 to 8 digits in length.)

NOTE: There is a 10 second programming time limit between each step. If RegulatorR times out or an error is made, programming will end and you will need to enter programming mode again.

1. Place RegulatorR into programming mode (See "**RegulatorR Programming Mode**").
2. Press "1" and then press **ENTER**.
3. Type the current user code and press **ENTER**.

4. Type the new user code and press **ENTER**.
5. Retype the new user code and press **ENTER**.
6. If successful, the **green** indicator light will illuminate for 1 second.

Continue programming or **press 9** and then **ENTER** to exit programming.

NOTE: Self Lock Regulator operates with a fixed code. **Changing the Supervisor Code**

To change the supervisor code, refer "**Regulator Programming Mode**" to enter programming mode. The factory default supervisor code is **12345** (supervisor codes must be 4 to 8 digits in length.)

NOTE: There is a 10 second programming time limit between each step. If Regulator times out or an error is made, programming will end and you will need to enter programming mode again.

1. Place Regulator into programming mode (See "**Regulator Programming Mode**").
2. Press "**2**" and then press **ENTER**.
3. Type the current supervisor code and press **ENTER**.
4. Type the new supervisor code and press **ENTER**.
5. Retype the new supervisor code and press **ENTER**.
6. If successful, the **green** indicator light will illuminate for 1 second.

Continue programming or **press 9** and then **ENTER** to exit programming.

Reset Instructions

Regulator[®] features a reset code which will restore the default supervisor code and user code. Regulator was shipped with reset stickers; one attached to the back of the Regulator and an extra sticker that should be saved in a safe and secure location for future reference.

NOTE: To restore the factory default supervisor and user code, type the "**Reset Code**" assigned to the Regulator and press **ENTER**.

Programming Security Timeout ON / Security Timeout OFF

WARNING: Security Timeout ON will disable the Regulator from use for 5 minutes when 5 invalid attempts to unlock Regulator occur. During this time, Regulator will not unlock with either the valid user or supervisor code.

To change the Regulator to Security Timeout ON or OFF, refer "**Regulator Programming Mode**" to enter programming mode.

NOTE: There is a 10 second programming time limit between each step. If Regulator times out or an error is made, programming will end and you will need to enter programming mode again.

1. Place Regulator into programming mode (See "**Regulator Programming Mode**").
2. To turn **Security Timeout ON**, type "**11**" and then press **ENTER**.

OR

1. To turn **Security Timeout OFF**, type "**10**" and then press **ENTER**.
2. If successful, the **green** indicator light will illuminate for 1 second.

Continue programming or **press 9** and then **ENTER** to exit programming.

Reset User Code to Factory Default 1234

To reset the **User Code** to the factory default **1234**, refer "**Regulator Programming Mode**" to enter programming mode.

NOTE: There is a 10 second programming time limit between each step. If Regulator times out or an error is made, programming will end and you will need to enter programming mode again.

1. Place Regulator into programming mode (See "**Regulator Programming Mode**").
2. Type "**8888**" and then press the **ENTER** button.
3. If successful, the **green** indicator light will illuminate for 1 second.

Continue programming or **press 9** and then **ENTER** to exit programming.

Add User Without Specifying Memory Slot

The **Supervisor Code** memory location is called **Slot 01** and the first **User Code** is called Slot 02.

- Any additional user codes reside in memory slots 03-20.
- To add a user code without specifying a memory slot (03 -20), refer to "**RegulatoR Programming Mode**" to enter programming mode.

User codes must be 4 to 8 digits in length and **must** be different from all existing codes (including Slot 01 Supervisor & Slot 02 User codes).

NOTE: There is a 10 second programming time limit between each step. If RegulatoR times out or an error is made, programming will end and you will need to enter programming mode again.

1. Place RegulatoR into programming mode (See "**RegulatoR Programming Mode**").
2. Press "410" and then press **ENTER**.
3. Type the new user code and press **ENTER**.
4. Retype the new user code and press **ENTER**.
5. If successful, the **yellow** indicator light will illuminate.

Continue programming or press **9** and then **ENTER** to exit programming.

Add User to Specific Memory Slot (03 – 20)

The **Supervisor Code** memory location is called **Slot 01** and the first **User Code** is called Slot 02.

- Any additional user codes reside in memory slots 03-20.
- To add a user code to a specific memory slot (03 -20), refer to "**RegulatoR Programming Mode**" to enter programming mode.

User codes must be 4 to 8 digits in length and **must** be different from all existing codes (including Slot 01 Supervisor & Slot 02 User codes).

NOTE: There is a 10 second programming time limit between each step. If RegulatoR times out or an error is made, programming will end and you will need to enter programming mode again.

1. Place RegulatoR into programming mode (See "**RegulatoR Programming Mode**").
2. Press "411" and then press **ENTER**.
3. Type memory slot to be added followed by **ENTER** (for memory slots 03 - 09, you must type leading "0" – e.g. "07" for slot 7).
4. Retype memory slot to be added followed by **ENTER**.
5. Type the new user code and press **ENTER**.
6. Retype the new user code and press **ENTER**.
7. If successful, the **yellow** indicator light will illuminate.

Continue programming or press **9** and then **ENTER** to exit programming.

Delete User Without Specifying Memory Slot

The **Supervisor Code** memory location is called **Slot 01** and the first **User Code** is called Slot 02.

- The additional user codes reside in memory slots 03-20.
- To delete a user code without specifying a memory slot (03 -20), refer to "**RegulatoR Programming Mode**" to enter programming mode.

NOTE: There is a 10 second programming time limit between each step. If RegulatoR times out or an error is made, programming will end and you will need to enter programming mode again.

1. Place RegulatoR into programming mode (See "**RegulatoR Programming Mode**").
2. Press "400" and then press **ENTER**.
3. Type the user code to be deleted and press **ENTER**.
4. Retype the user code to be deleted and press **ENTER**.
5. If successful, the **yellow** indicator light will illuminate.

Continue programming or press **9** and then **ENTER** to exit programming.

FLEXLINE ELECTRONIC PIN LOCK — INSTRUCTIONS FOR USE

Delete User From Specific Memory Slot (03 – 20)

The Supervisor Code memory location is called Slot 01 and the first User Code is called Slot 02.

- The additional user codes reside in memory slots 03-20.
- To delete a user code from a specific memory slot (03 -20), refer "**RegulatoR Programming Mode**" to enter programming mode.

NOTE: There is a 10 second programming time limit between each step. If RegulatoR times out or an error is made, programming will end and you will need to enter programming mode again.

1. Place RegulatoR into programming mode (See "**RegulatoR Programming Mode**").
2. Press "401" and then press ENTER.
3. Type memory slot to be vacated followed by ENTER (for memory slots 03 - 09, you must type leading "0" – e.g. "07" for slot 7).
4. Retype memory slot to be vacated followed by ENTER.
5. If successful, the **yellow** indicator light will illuminate.

Continue programming or press 9 and then ENTER to exit programming.

THIS PAGE IS KEPT INTENTIONALLY BLANK

* Save this document for future application, load rating and/or safety reference.

© 2019 InterMetro Industries Corporation, Wilkes-Barre, PA 18705
L01-655 REV 01/20

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