CaseVue™

Case Carts / Chariots / Carros de almacenamiento

Doors	Portes	Puertas	2
Casters	Roulettes	Ruedas giratorias	3
Fifth Wheel	Cinquième roue	Quinta rueda	4
Wire & Solid Shelf	Étagères métalliques pleine et grillagée	Estantería de alambre y estante sólido	4
Swing Handle	Poignée pivotante	Manija oscilante	5
Flag (Status Indicator)	Drapeaux (indicateur d'état)	Marca (indicador de estado)	6
Paper Holder	Porte papier	Soporte para papel	6
Optional Door Seal (Gasket)	Joint de porte en option (joint d'étanchéité)	Sello opcional de la puerta (junta)	6
Weight Capacity	Capacité de poids	Capacidad de peso	7
Caster / Cart Maintenance Recommendations	Recommandations d'entretien des des roulettes/du chariot	Recomendaciones de mantenimiento de las ruedas giratorias/los carros	7
Cart Cleaning / Sterilization Guide For Manual Cleaning	Guide de nettoyage/de stérilisation du chariot pour un nettoyage manuel	Guía de limpieza/esterilización del carro para la limpieza manual	7
Automated Cart Wash Parameters	Paramètres de lavage automatisé du chariot	Parámetros de lavado automatizado del carro	7
Replacement Parts	Pièces de rechange	Piezas de repuesto	8
Replacement Parts Diagram	Schéma des pièces de rechange	Diagrama de las piezas de repuesto	9
Application of Silicone	Application de silicone	Aplicación de silicona	10

Doors / Portes/ Puertas

Door latch / Loquet / Pestillo de la puerta

To open, lift up on latch paddle to open the door(s) (Fig. A). On double door units, the right door must be opened before the left door. To close, swing door(s) back into cart. Left door must be closed before the right door.

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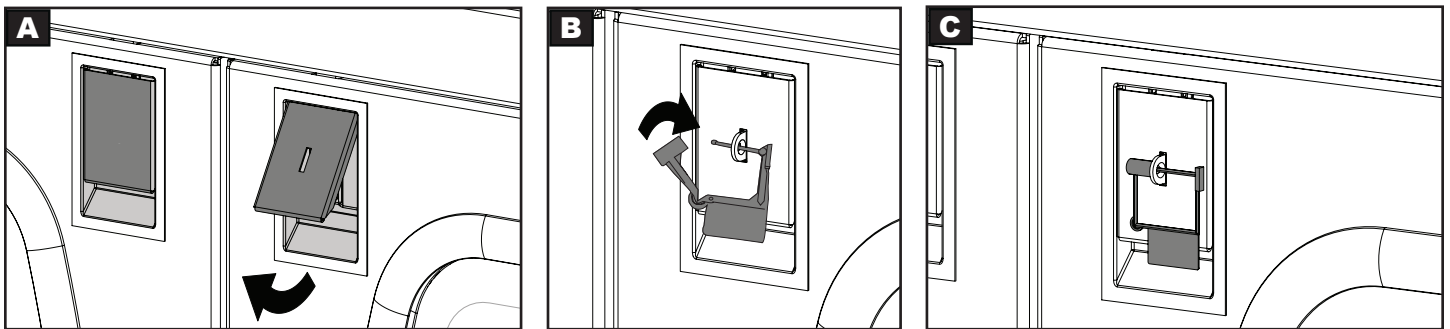
Install optional Metro Security Seal (sold separately) shown in Fig B into latch hasp as shown in Fig C. To gain access to the cart, simply pull the security seal.

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
Side Door Catch / Targette de porte latérale / Traba lateral de la puerta


To lock the door(s) in the open position, simply push the door(s) in against side of unit. To disengage, pull up door keeper catch as shown in (Figure B) and Pull Door outward.

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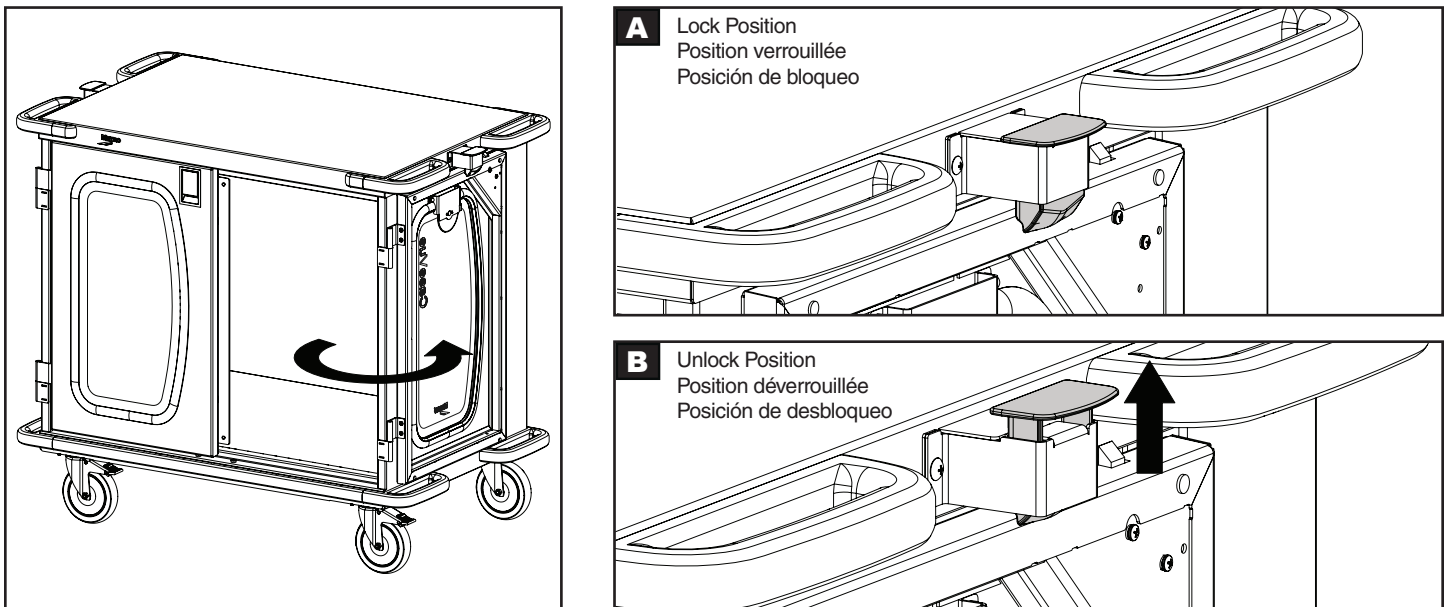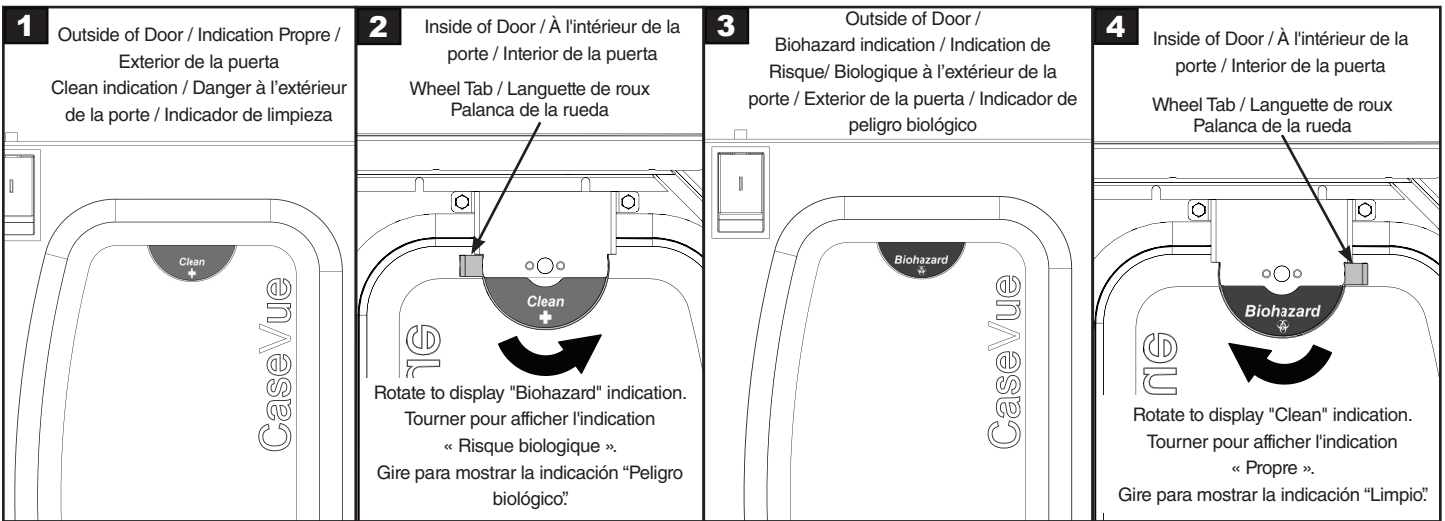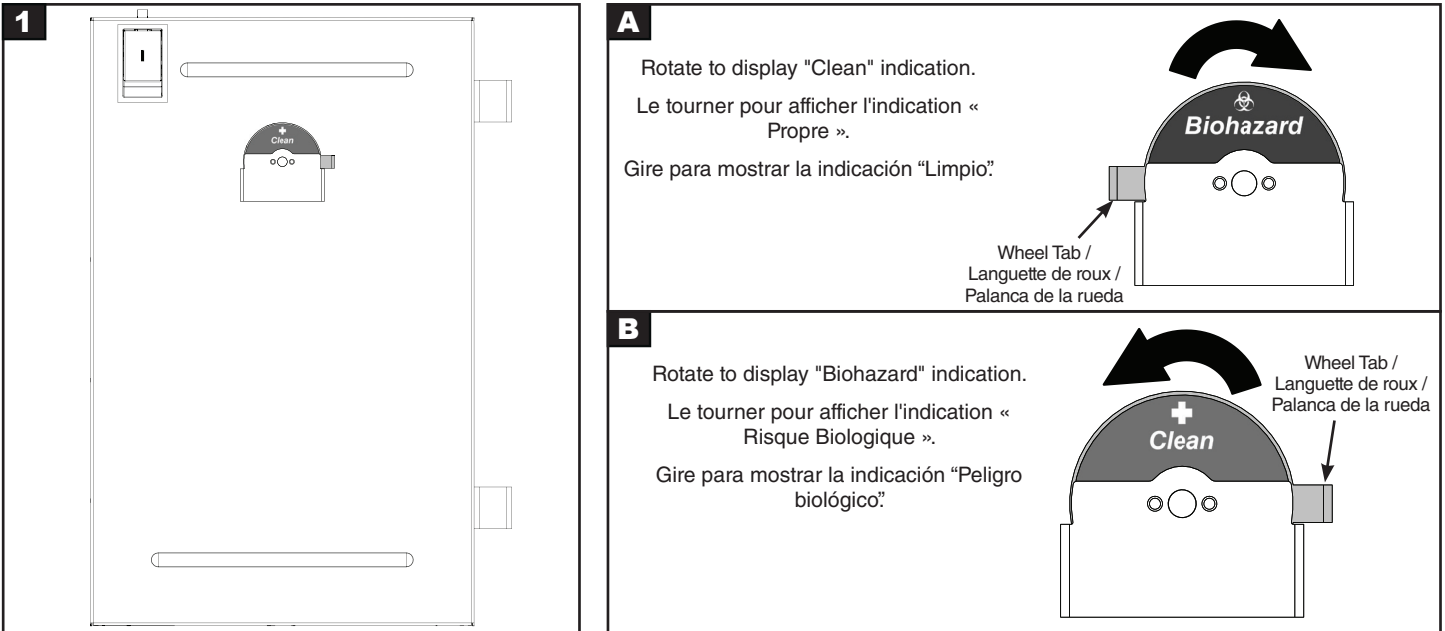
Casters / Roulettes / Ruedas giratorias

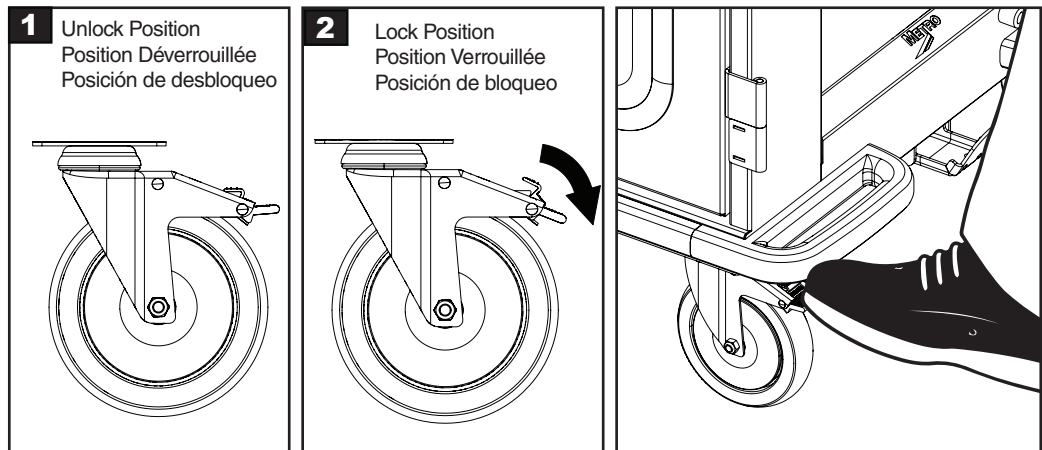
Front Casters: Total Lock
Rear Casters: Swivel Lock or Swivel
Note: Swivel casters are provided on rear of carts equipped with 5th wheel.

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





Optional 5th wheel steering assist assures maximum control in transit.

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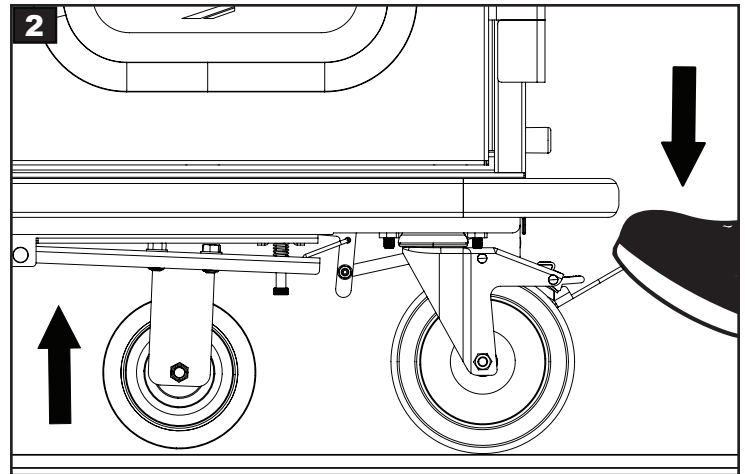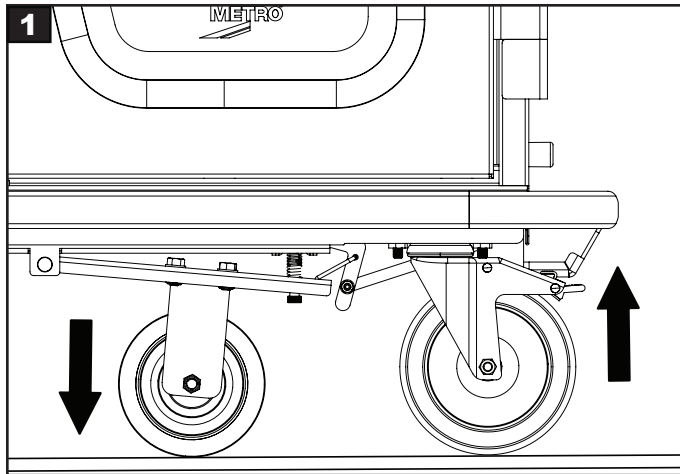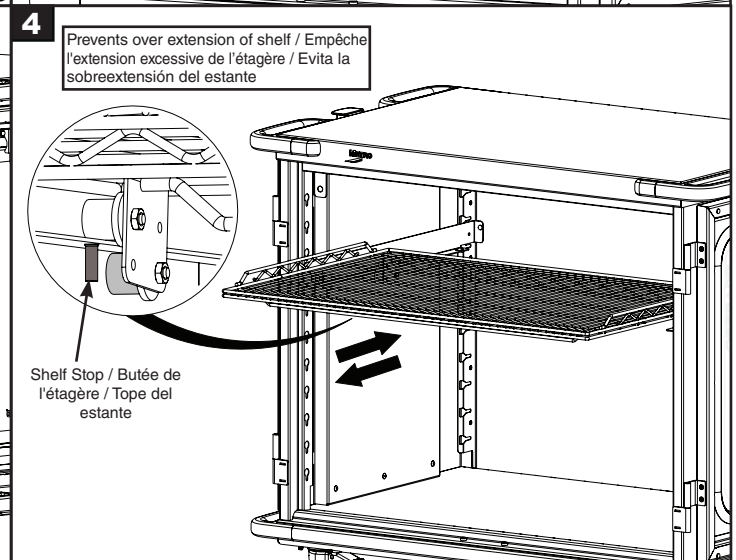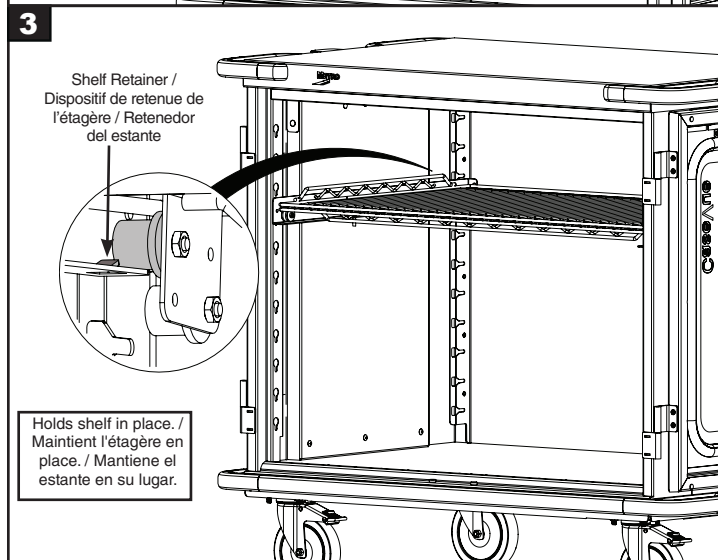
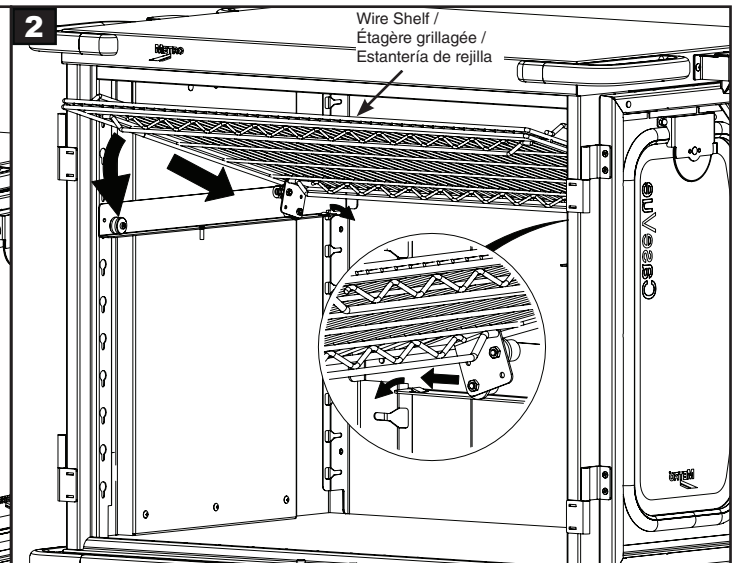
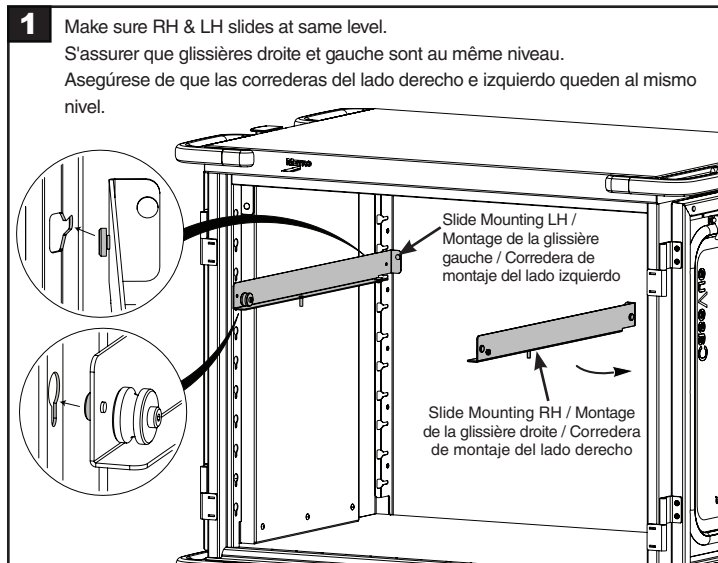
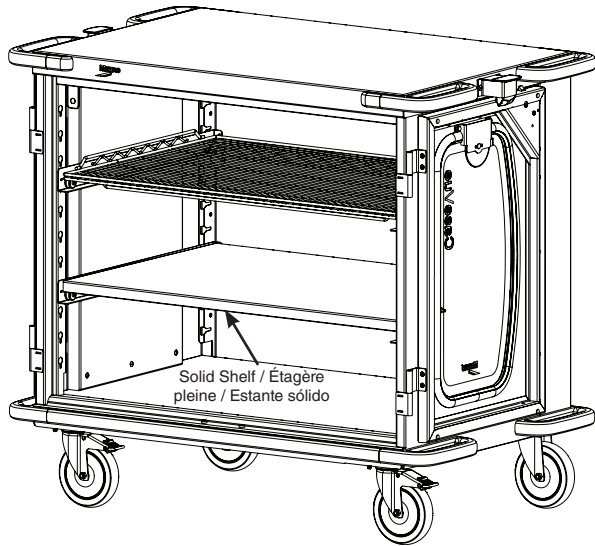
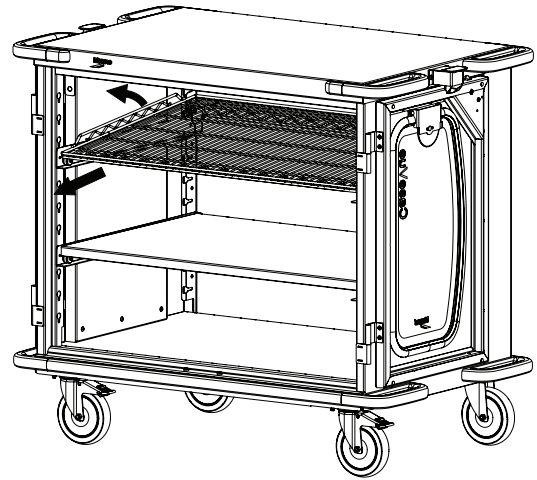
Provide adjustable height, added clearance for towing, and more leverage for steering.

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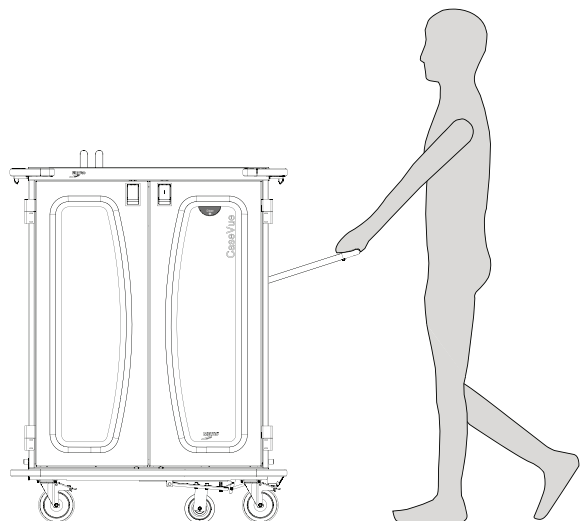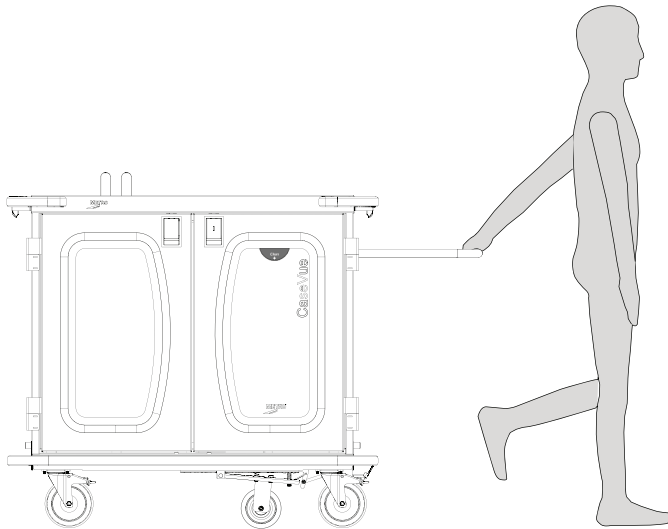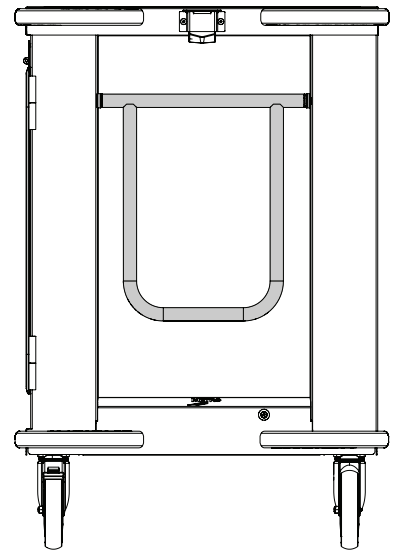Identify cart status from a distance or among the entire fleet. Customize a color code process to maximize efficiency.

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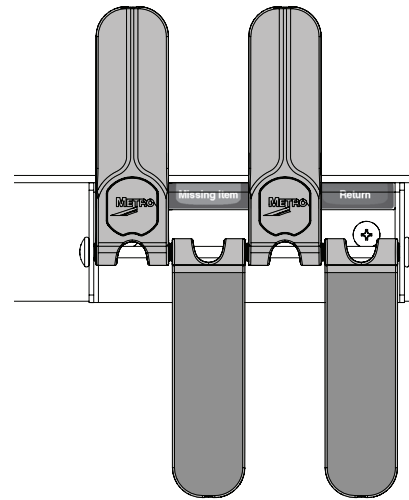
Paper Holder / Porte papier / Soporte para papel

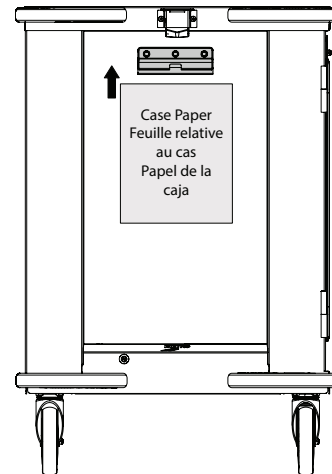
Conveniently holds routing slips and/or pick lists.

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CASEVUE CART AUTOMATED CART WASH PARAMETERS

1. The Cart Wash Chamber must be non-pressurized.
2. Water temperatures in the range of 140°F (60°C) to 180°F (82.2°C) are allowed with an exposure period not to exceed 30 minutes.
3. Hot air drying is allowed with the temperatures not to exceed 200°F (93.3°C) and an exposure period not to exceed 15 minutes.
4. Steam sanitation is allowed with the temperature not to exceed 212°F (100°C) and an exposure period not to exceed 15 minutes.
5. Phosphoric acid based sanitation is allowed with the concentration of phosphoric acid not to exceed 10% by volume.
6. Sodium hydroxide based sanitation is allowed with the concentration of sodium hydroxide not to exceed 10% by volume.
7. Potassium hydroxide based sanitation is allowed with the concentration of sodium hydroxide not to exceed 10% by volume.
8. Sodium hypochlorite, chlorine, chlorine dioxide and/or bleach based sanitation MUST be avoided.

WEIGHT CAPACITY / CAPACITÉ DE POIDS / CAPACIDAD DE PESO

Cart weight: 400 lbs maximum equally distributed
Shelf weight: 125 lbs maximum equally distributed

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CASEVUE CART CLEANING/ STERILIZATION GUIDE FOR MANUAL CLEANING

DISINFECTANT/ CLEANER	GROUP	CART BODY & SHELVES INCLUDING SILICONE RUBBER JOINT SEALANT	CART BUMPERS	CASTERS	CLEAR DOORS
Spor-Klenz (Ready To Use)	Peracetic Acid Based	YES	YES	YES	YES
Process Vesphene II st	Non-Alkaline Phenolic Based	YES	YES	YES	YES
Vesphene II se	Alkaline Phenolic Based	YES	YES	YES	YES
Process LPHst	Non-Alkaline Phenolic Based	YES	YES	YES	YES
LPH se	Non-Alkaline Phenolic Based	YES	YES	YES	YES
Virex 256	Quaternary Ammonium Based	YES	YES	YES	NO
Clidox-S (Chlorine Dioxide)	Chlorine Dioxide Based	NO	YES	NO	YES
Virkon-S	Potassium Salt Based	YES	YES	YES	YES
Isopropyl Alcohol (70%)	2-Propanol	YES	YES	NO	YES
Formula 409, Fantastik	Quaternary Ammonium Based	YES	YES	YES	NO
Bleach/Chlorine/ Sodium Hypochlorite	Halogen	NO	NO	NO	NO
Joy, Palmolive, Mr. Clean, etc.	Misc. Surfactant Detergents	YES	YES	YES	YES

YES ratings are based on adherence to the manufacturer's recommended dilution ratio for the cleaner/disinfectant concentrate being used (where applicable), at ambient temperatures not exceeding 72° F (22.2° C), and with intermittent exposure times not exceeding 15 minutes each exposure.

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
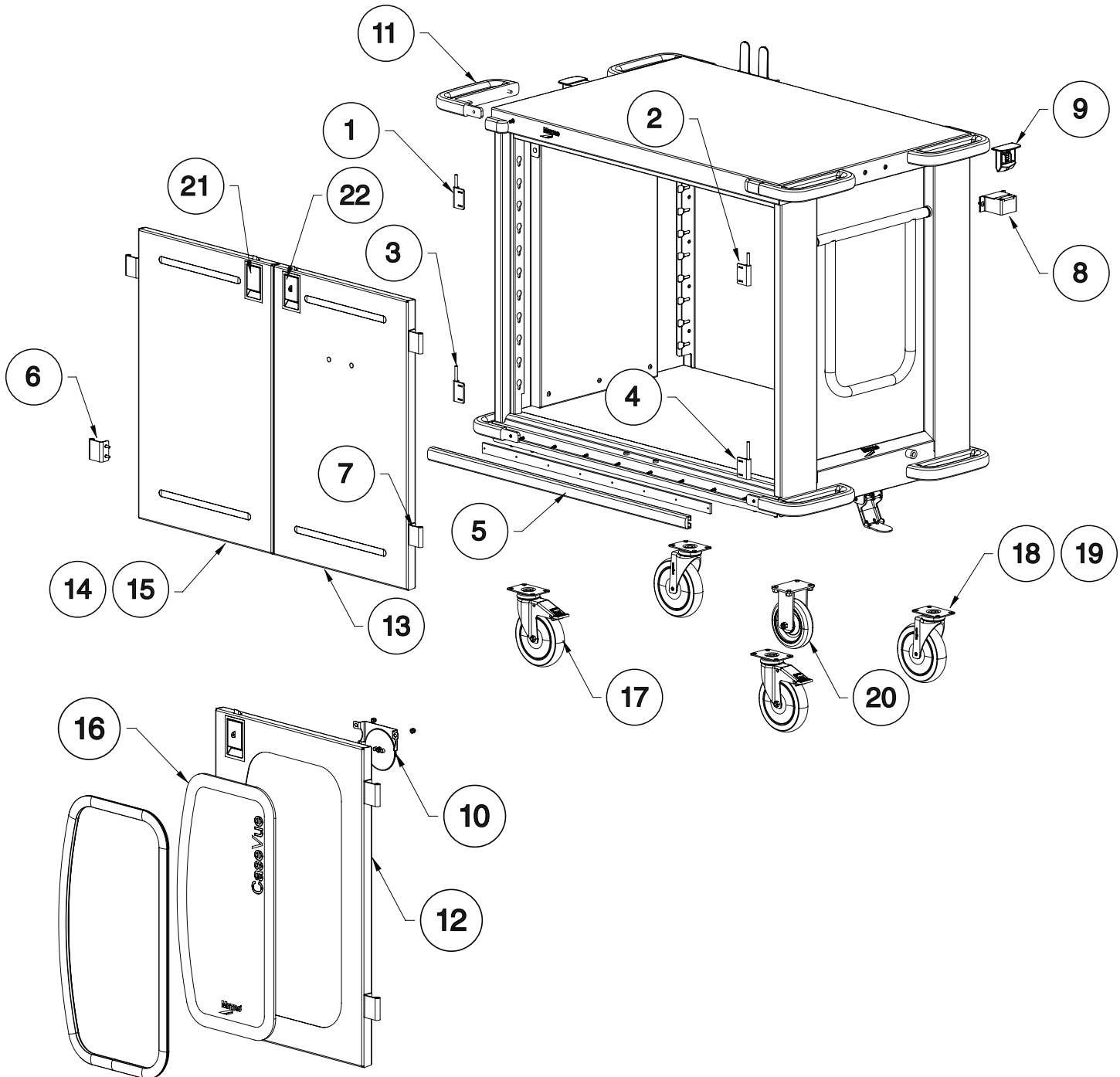
Application of Silicone / Application de silicone / Aplicación de silicona

If cart washing is frequent and/or prolonged, the silicone sealed joints should also be periodically inspected to insure that the integrity and adhesion of the silicon seal to the metal substrate is acceptable and remains capable of maintaining a watertight seal. If visible degradation of the silicone seal becomes apparent, the existing sealant should be removed and replaced with Dow Corning® Silastic #733 Multi-Purpose Sealant or equivalent.

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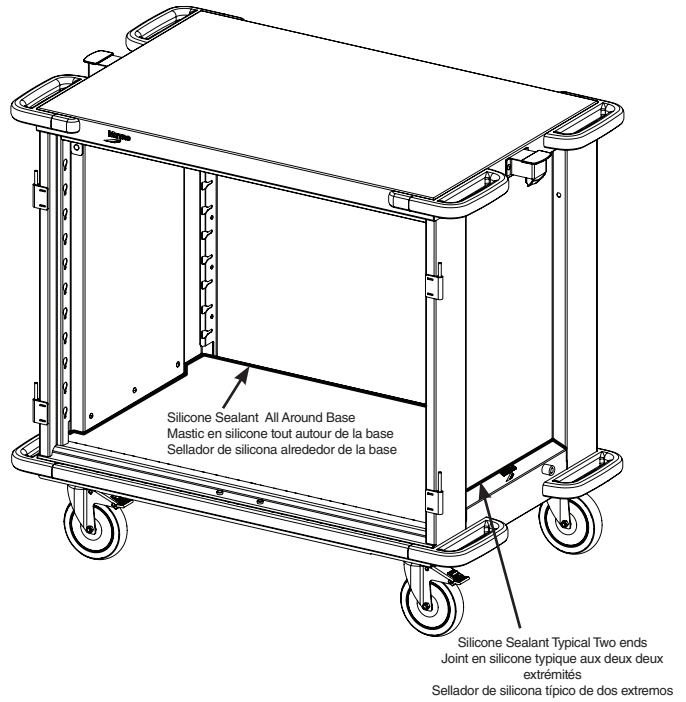* Save this document for future application, load rating and/or safety reference.

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